

Installation Guide

L-Bracket Attachment for Devices with VESA Pattern (Refer to qualified personnel)

Purpose:


To describe the installation of Device with VESA Pattern onto L-Bracket



Parts Reference

The following parts and hardware are included in this installation kit (see illustration for part; hardware not shown):

Item #	Description	Qty
1	VESA Plate	1
2	Teflon Washer	1
3	Fender Washer	1
4	Knob	1
5	M4 x 8mm Cheese Head	4
6	M4 x 10mm Flat Head Machine Screw (FHMS)	4
7	M4 x 12mm Flat Head Machine Screw (FHMS)	4
8	M4 x 16mm Flat Head Machine Screw (FHMS)	4

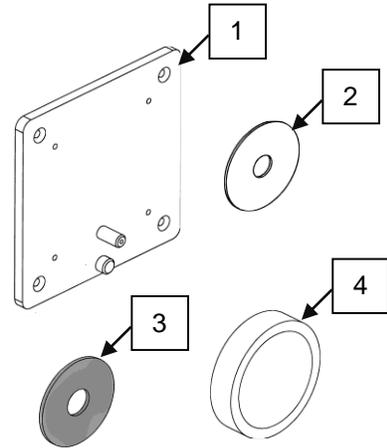

Tools Required:


- Phillips Screwdriver (not provided)




WARNING: This mounting kit provides an assortment of hardware for a wide variety of display-mounting applications. It is the responsibility of the installer of this product to ensure that all screws used to mount the display have a minimum thread engagement of four (4) 360° turns into threaded inserts in rear of display and into threaded inserts in VESA Mounting Adapter. It is also the responsibility of the installer to ensure that screws are not inserted too far into the display, causing damage to internal components. Failure to adhere to this warning could result in damage to equipment or injury to patients or personnel.

Installation of Device onto L-Bracket Attachment

Step 1

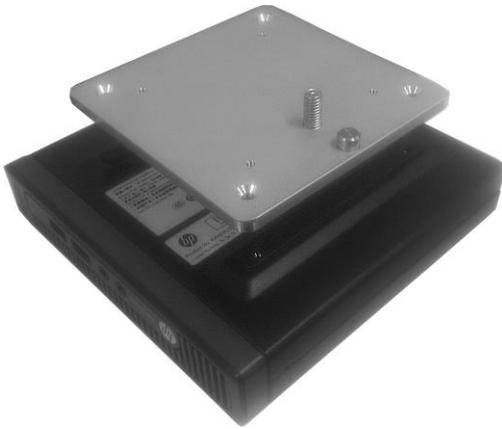


Step 2

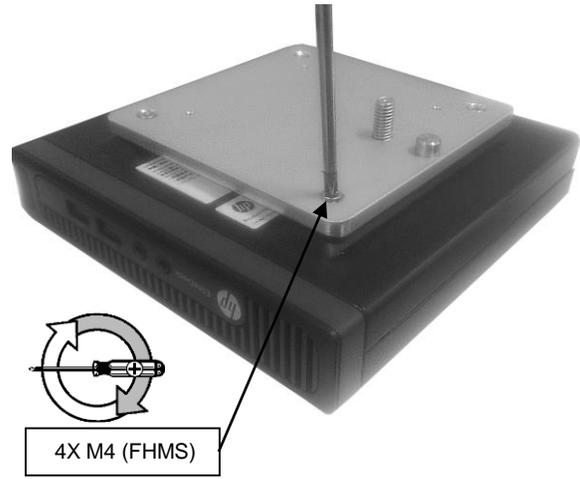


Step 3



Step 4

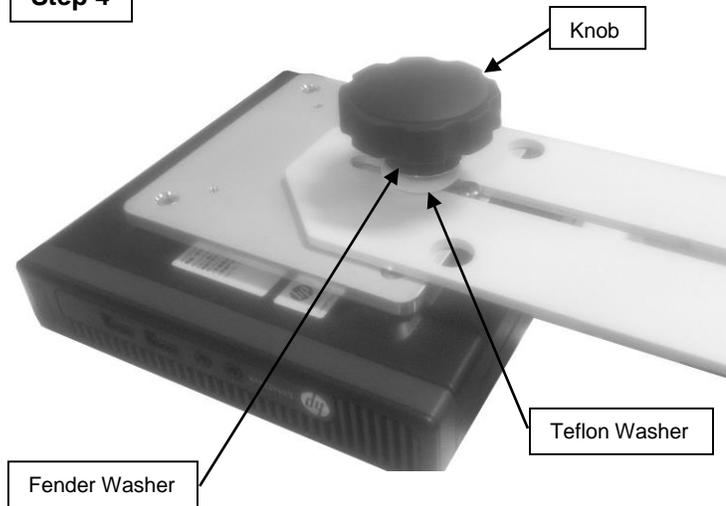

Installation of Wyse Thin Client Bracket onto L-Bracket Attachment

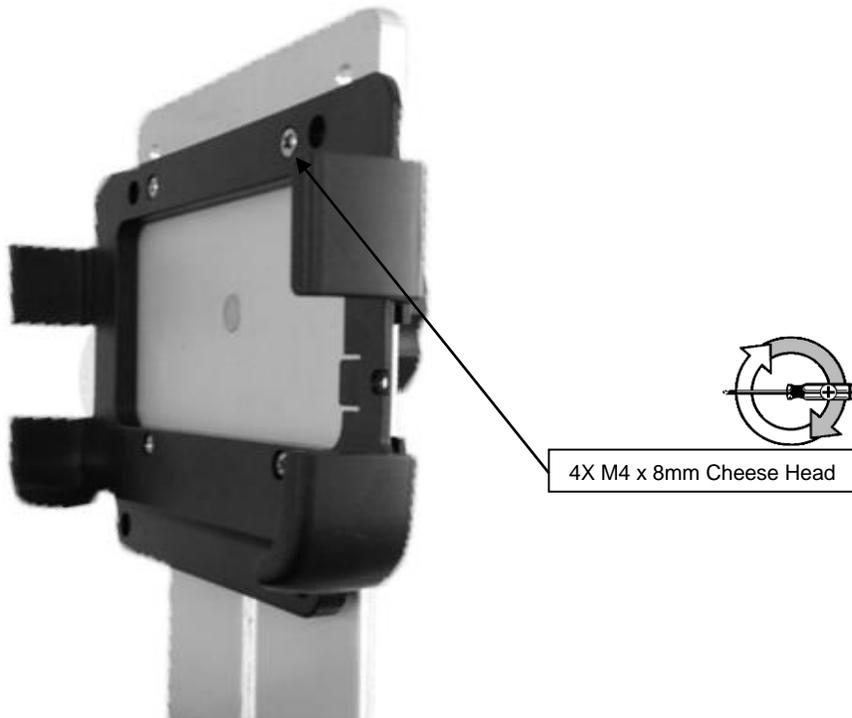

Routine Maintenance

Periodically inspect all fasteners associated with the mounting assembly. Tighten or Adjust as necessary for optimal operation and safety.

Cleaning the Mounting Assembly

1. The mounting assembly may be cleaned with most mild, non-abrasive solutions commonly used in the hospital environment (e.g. diluted bleach, ammonia, or alcohol solutions).
2. The surface finish will be permanently damaged by strong chemicals and solvents such as acetone or trichloroethylene.
3. Steel wool or other abrasive material should never be used.
4. Damaged caused by the use of unapproved substances or processes will not be warranted. We recommend testing of any cleaning solution on a small area of the mounting assembly that is not visible to verify compatibility.
5. Never submerge or allow liquids to enter the mounting assembly. Wipe any cleaning agents off the mounting assembly immediately using a water-dampened cloth. Dry the assembly thoroughly after cleaning.

CAUTION: GCX makes no claims regarding the efficacy of the listed chemicals or processes as a means for controlling infection. Consult your hospital's infection control officer or epidemiologist. To clean or sterilize mounted devices or accessory equipment, refer to the specific instructions delivered with those products.