

Installation Guide

Stack Mount Kit for Philips G5

The purpose of this guide is to:


- 1. Describe attachment of Mount to G5 (page 1).
- 2. Describe installation of 12" Post on Mount (page 2).



Parts Reference

The following parts and hardware are included with this installation kit (hardware not shown):

| Item # | Description | Qty |
|--------|--|-----|
| 1 | Stack Mount (6" Post attached) | 1 |
| 2 | M6 x 12mm Flat Head Machine Screw (FHMS) | 3 |
| 3 | 12" Post | 1 |

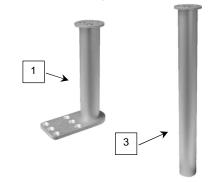


Tools required: Phillips screwdriver (not provided).

Installation Note: The Stack Mount is pre-configured at the factory with a 6" Post. See page 2 if the 12" Post is required.


Attaching Stack Mount to G5

1. Fasten Mount to G5 with three (3) M6 x 12mm FHMS as shown below.

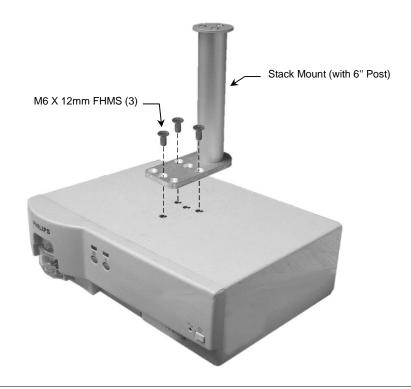

Mounting the FMS on the Post

Mount FMS according to Philips mounting instructions. Typical application shown below.

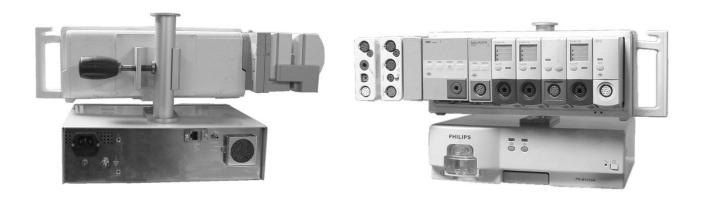

Installing the 12" Post on the Mount

- 1. Unscrew three (3) pan head screws from the bottom of the Mount and remove the 6" Post. Do not discard screws.
- 2. Fasten 12" Post to Mount with screws removed in step 1. Follow "Attaching Stack Mount to G5" on page 1.

