


Installation Guide

Slide-In Mounting Plate for Top of VHRC/RC Column (Refer to qualified personnel)

Purpose:

To describe the installation of the Slide-In Mounting Plate into the top of the VHRC/RC Column

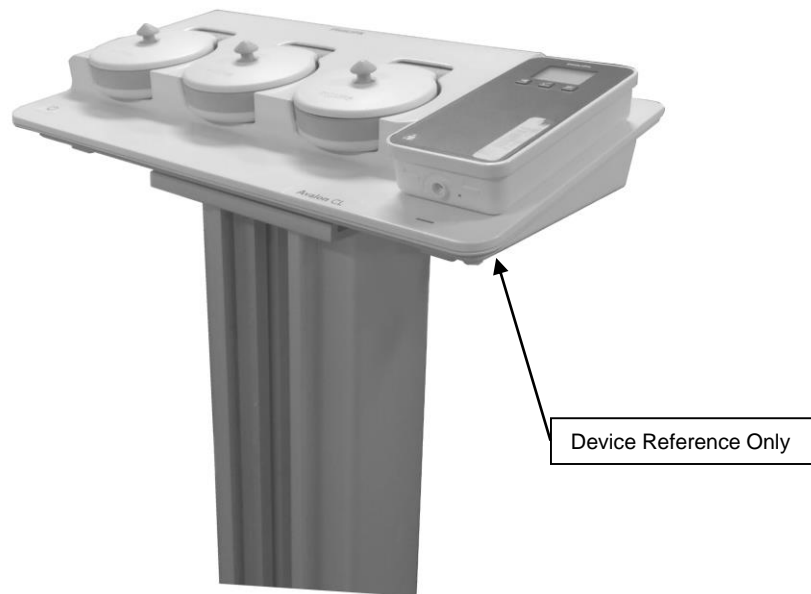


Parts Reference

The following parts and hardware are included in this installation kit:

Item #	Description	Qty
1	Slide-In Mounting Plate	1
2	5/16-18 x 1" Flat head Machine Screw (FHMS)	4

Tools Required:


- Phillips Screwdriver (not provided)

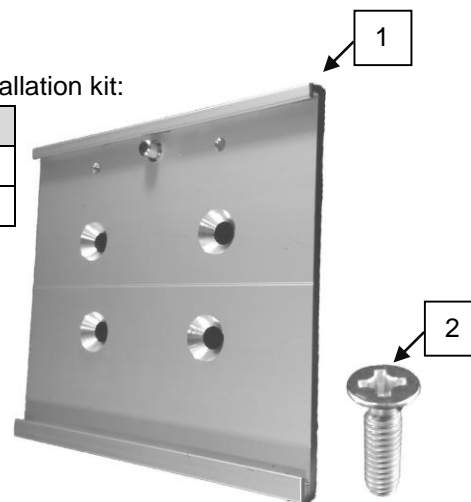


- For installation of VESA adapter plate onto device, refer to separate instructions

Attachment of the Slide-In Mounting Plate to the Top of VHRC/RC Column

1. Remove and dispose of the Top Cover and screws placed on the Column.

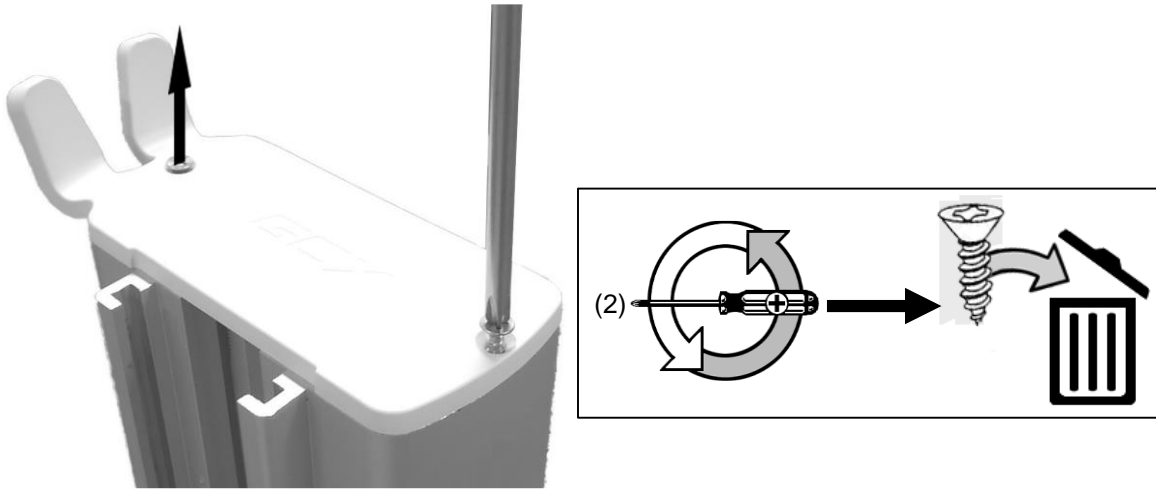


2. Place the Slide-In Mounting Plate on top of the Column with the Plunger facing the back of the Column, and fasten with four (4) 5/16-18 x 1" Flat Head Machine Screws (FHMS).

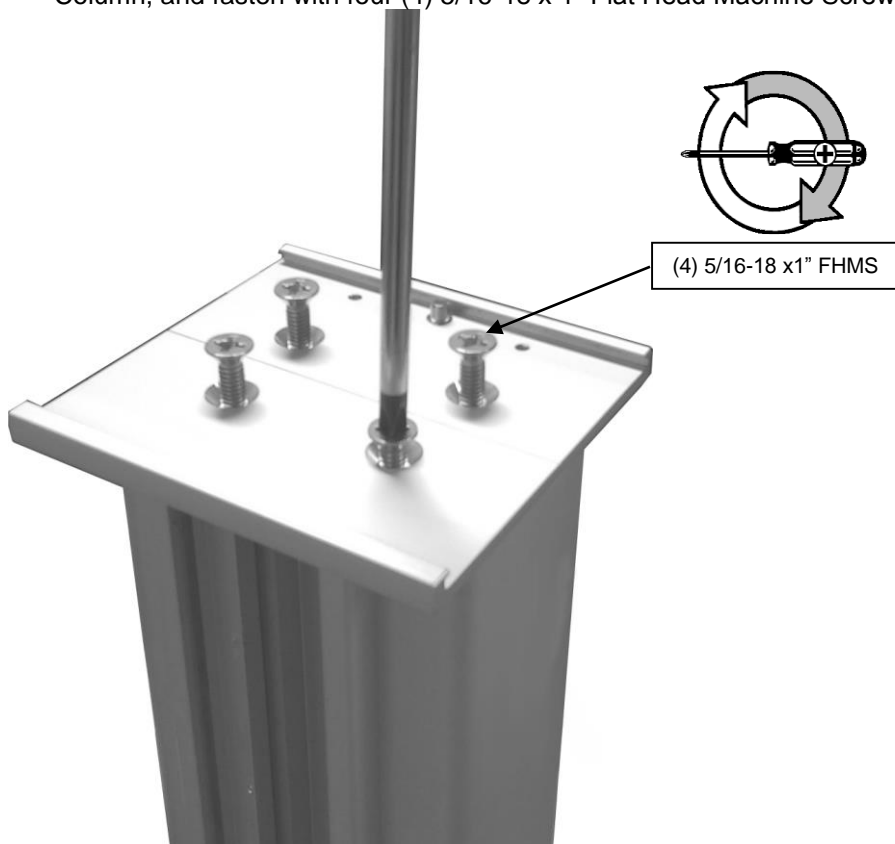


3. Slide the device with the VESA plate into the Slide-In Mounting Plate.



4. Fasten the device by pulling the Plunger on the Slide-In Mounting Plate and releasing when the device is in place.

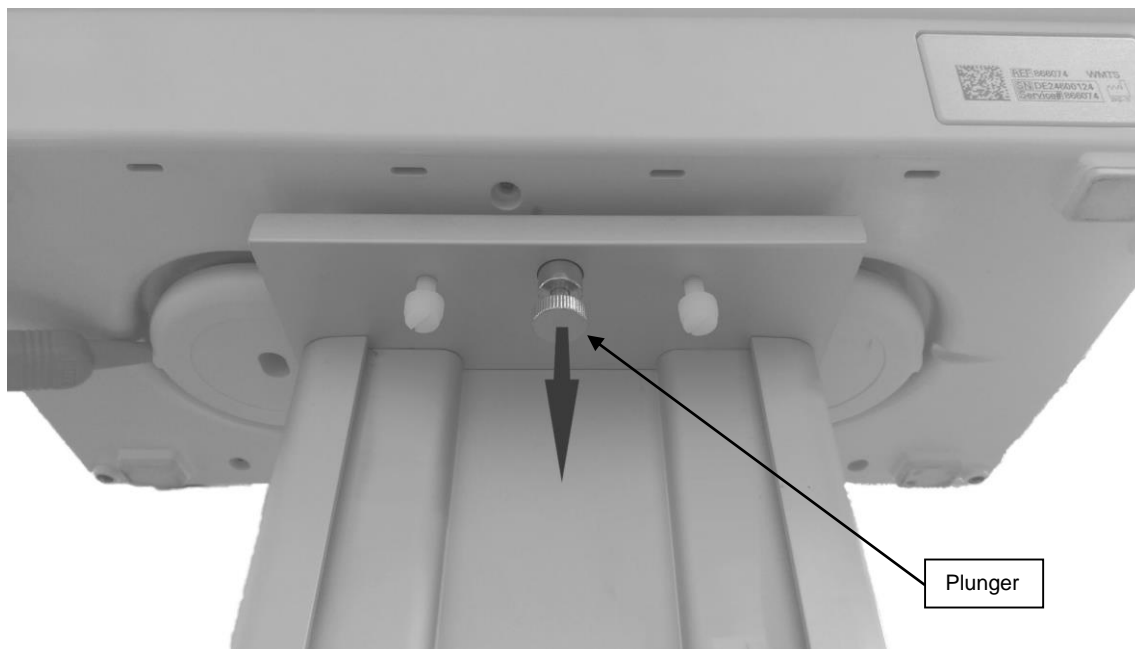

5. Secure the plate by tightening the two (2) Thumb Screws on the Slide-In Mounting Plate.

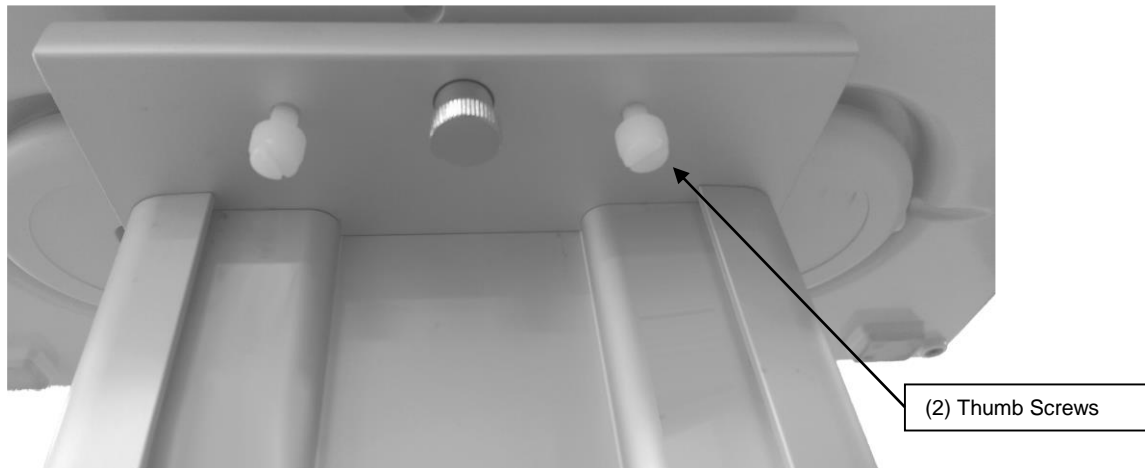

Routine Maintenance

Periodically inspect all fasteners associated with the mounting assembly. Tighten or Adjust as necessary for optimal operation and safety.

Cleaning the Mounting Assembly

1. The mounting assembly may be cleaned with most mild, non-abrasive solutions commonly used in the hospital environment (e.g. diluted bleach, ammonia, or alcohol solutions).
2. The surface finish will be permanently damaged by strong chemicals and solvents such as acetone or trichloroethylene.
3. Steel wool or other abrasive material should never be used.
4. Damaged caused by the use of unapproved substances or processes will not be warranted. We recommend testing of any cleaning solution on a small area of the mounting assembly that is not visible to verify compatibility.
5. Never submerge or allow liquids to enter the mounting assembly. Wipe any cleaning agents off the mounting assembly immediately using a water-dampened cloth. Dry the assembly thoroughly after cleaning.

CAUTION: GCX makes no claims regarding the efficacy of the listed chemicals or processes as a means for controlling infection. Consult your hospital's infection control officer or epidemiologist. To clean or sterilize mounted devices or accessory equipment, refer to the specific instructions delivered with those products.