

## GCX Mounting Assembly Installation Guide

### Flush Wall Mount Power Strip with Coiled Cord Assembly (Tripp-Lite UL 1363A Rating)

*The purpose of this guide is to describe mounting of the Power Strip with Coiled Cord on GCX VHRC Carts.*

#### Parts Reference

The following parts and hardware will be used in this procedure:


| Item # | Description                          | Qty |
|--------|--------------------------------------|-----|
| 1      | Power Strip – Channel Slide Assembly | 1   |
| 2      | 1/8" Hex Key                         | 1   |



**Tools Required:** 1/8" hex key (provided).

#### Mounting Power Strip in Channel

1. Insert Slide (attached to rear of Power Strip) in Channel and slide Power Strip to desired mounting position.
2. Using 1/8" hex key provided tighten the socket set screw at the lower right corner of the Slide to secure position.

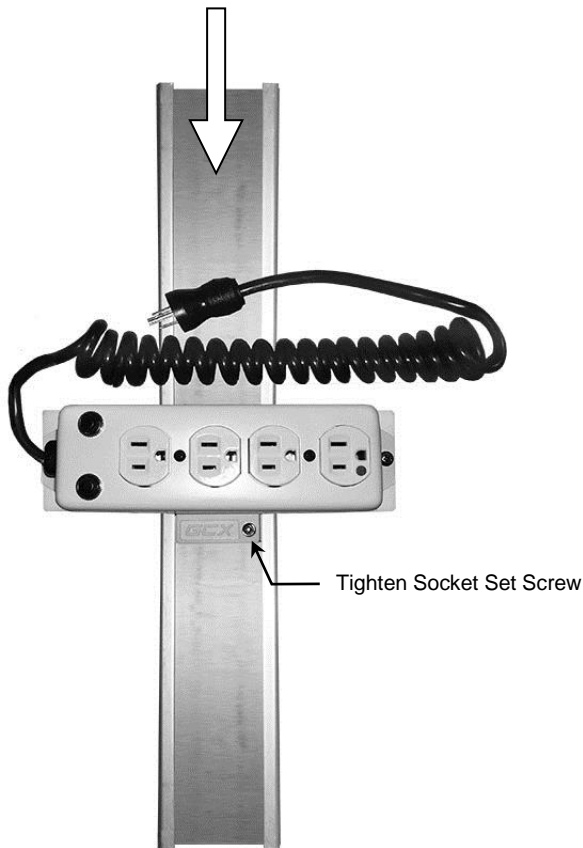

**Cord Storage on VHRC Cart**  
Coiled Cord can be stored during transport by utilizing molded hook at top of VHRC column.