

GCX Mounting Assembly Installation Guide


Dual Utility Hook

The purpose of this guide is to describe attachment of the Dual Utility Hook to Support Arm.

Parts Reference

The parts list below includes only the parts and hardware that will be used in this installation procedure (see photo for parts; hardware not shown).

Item #	Description	Qty
1	Dual Utility Hook	1
2	1.980" x .250" - #8-32 Stand Off	1
3	#8-32x1/4"PHMS-Patch	2

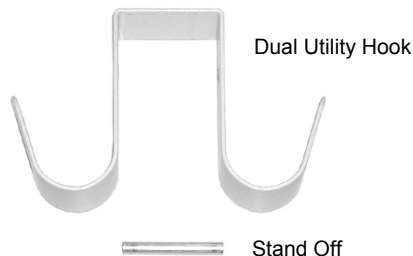

Tools Required: Phillips screwdriver.

Attaching the Dual Utility Hook

1. Place the Dual Utility Hook over the Support Arm to the desired position. Put the Stand Off underneath the arm and use the supplied #8-32x1/4"PHMS (2) to tighten the hook to the arm.

