

Installation Guide

Channel Cover

The Channel Cover provides an aesthetic cover and may be used for cable management along the Channel.

The purpose of this guide is to describe installation of Channel Cover in channel.

Parts Reference

The following parts will be used in this procedure:

Item #	Description	Qty
1	Channel Cover, 16"	1




Channel Cover

Tools Required: Scissors, utility knife or similar cutting tool.

Installing the Channel Cover

1. Cut Channel Cover with scissors for a custom fit (below left). The back side of the Cover is scored in sections for easier cutting, or may be cut as required anywhere along the Cover. **Cable Management:** Scored notches are also provided for routing cables through the side of Cover if required. Using a utility knife, or similar cutting tool, cut along the scored line to create a cable-routing notch (below center and right).

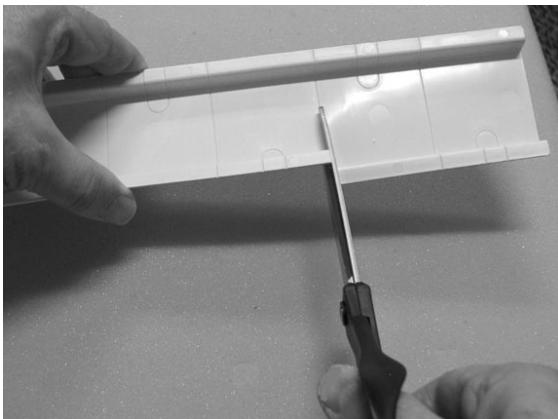

2. Press one side of the Cover into channel, routing cables behind Cover and through notches. Squeeze Cover and insert other side into channel.

